FLACONE 10 ML - 10 ML MULTI DOSE FLASK

COMPOSIZIONE/COMPOSITION: Purified Water, *Sea Water, Jacobaea Maritima Extract, Calendula Officinalis Flower Water, Centaurea Cyanus Flower Water, Helichrysum Italicum Flower Water, Euphrasia Officinalis Flower/Leaf/Stem Water, Hydroxypropylmethylcellulose, Sodium Hyaluronate.

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EN ISOMAR® Occhi Eye Drops

ISOMAR® Occhi gives a quick relief in case of reddening, irritation, burning, itching and foreign bodies eyes sensations, lachrymation and fatigue due to outdoor agents such as wind and sun exposure, dusts, smog, pollens, sailiness, dry heat, air conditioning, prolonged computer/TV use and wearing contact lenses.

ISOMAR® Occhi contributes to relieve disorders caused by the contact with pollen, helping to alleviate the allergic symptoms.

ISOMAR® Occhi brings relief in case of insufficient lachrymal secretion and can be used by people wearing any type of contact lenses: for a better lenitive and lubricating action. It's advised to use **ISOMAR® Occhi** before wearing them.

INSTRUCTION FOR USE: Open the pack and, before administering for the first time, remove the protective seal. Turn upside down over the eye, holding the bottle between your index finger and thumb. Gradually increase the amount of pressure used to squeeze the bottle between finger and thumb until sufficient solution has accumulated at the tip, so as to enable the drop to fall. Instill 1-2 drops into the eye every 4-5 hours. Immediately after use, return the bottle to a vertical position and replace the cap. The remaining solution can be reused within 6 months after opening. **The innovative ISOMAR® Occhi cap is specially designed to ensure the sterility of the product even during use and the optimal dispensing.** It's recommended not to press hard on the closed vial to prevent drops to get out from the cap grid. You may need to apply greater pressure as the solution contained in the bottle is progressively depleted.

CONTRAINDICATIONS: Contraindicated in children, in women who are pregnant or breast-feeding.

PRECAUTIONS: **ISOMAR® Occhi** is suitable for ophthalmic use only and is not to be shared among patients. Remove the cap before application and replace immediately after use. Do not use the product if the pack appears to be damaged upon opening. Do not pierce/tamper with the bottle. Don't use the product if the pack is not intact. Don't use in case of incompatibility towards the ingredients of the product. Do not use concomitantly with other topical treatments. The device must not be used for cleaning and/or storing contact lenses. In the event of undesirable side effects, suspend treatment and consult a doctor. Don't touch the membrane of the conjunctiva with the top of the vial. Keep it far from children. Don't use the product after the expiry date on the package or bottle. The expiry date applies to an intact and properly stored product. Store in a cool, dry place, away from sources of heat and light, at a temperature between 4° and 25°C. Don't waste the vial in the environment after its use.

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