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EN ISOMAR® Occhi Eye Drops Plus

INDICATIONS FOR USE: Isomar Occhi Plus is especially indicated for the treatment of dry eye, providing relief from the burning sensation and irritation caused by the condition.

Isomar Occhi Plus, with its special formulation, helps to soothe the symptoms of dry eye. By means of the following actions:

1. Osmoprotectant properties: one of the main causes of dry eye is reduced or abnormal tear secretion due to a disturbance of the natural osmotic cellular balance. Isomar Occhi Plus contains the Isoeye Plus complex (Propylene Glycol, Erythritol, Betaine, L-Carnitine), which provides an osmoprotectant action, moisturises and protects the epithelial cells of the cornea from potential cell damage caused by hyperosmolarity.

2. Moisturising properties: hyaluronic acid and hypromellose stabilise the lacrimal film, helping to keep the ocular surface lubricated and moisturised.

3. Lubricant properties: Hyaluronic acid and hypromellose stabilise the lacrimal film, helping to keep the ocular surface lubricated and moisturised.

Isomar Occhi Plus provides relief from redness, stinging irritation, itching and the sensation of foreign bodies, as well as from tearing and fatigue induced by factors such as exposure to wind and sunlight, dust, pollen, smog, smoke, dry heat, air-conditioned environments, prolonged exposure to computer screens and contact lens wear. Isomar Occhi Plus can be used with all types of contact lenses. Application of Isomar Occhi Plus prior to fitting contact lenses is recommended for an enhanced lubricating and soothing action.

INSTRUCTIONS FOR USE: to use the single-dose vials correctly, follow the instructions below:

- open the protective packaging and detach a single-dose vial;
- open the single-dose vial by twisting the cap until it is released;
- tilt the head and, holding the lower eyelid gently with a finger, apply 1-2 drops of solution in the conjunctival sac, every 4-5 hours;
- close the eye for a few seconds.

When opened for the first time, the vial can be closed again using the cap and the remaining eye solution can be used over the next 12 hours. If required, the procedure can be repeated for the other eye.

INTENDED PATIENT POPULATION: the medical device is intended for users with no specialist knowledge by observing the manufacturer's instructions for use. The medical device is intended for use in home environments. The user is provided with all the necessary information for the correct and safe use of the device.

CONTRAINDICATIONS: the medical device is not suited for subjects with known hypersensitivity to the components in the formula.

WARNINGS: before use, carefully read the information provided by the manufacturer. Do not exceed the recommended dose. Discontinue treatment if any undesirable effect occurs. Do not use in conjunction with other topical treatments. Do not swallow. The product is intended for ophthalmic use only. Store inside the box in a dry location, away from sources of light and heat, at a temperature between 5°C and 25°C. Do not pierce or tamper with the packaging. Do not use the product if the package is opened or damaged. Do not use the product after the expiry date. The expiry date refers to the unopened and properly stored product. The product is intended for adult patients. There is no specific data available regarding the use of the product on children and pregnant women. Keep out of sight and reach of children. It is advisable to wash hands thoroughly before applying the solution and to not touch the tip of the dispenser with the fingers. During administration of the solution, make sure that the conjunctival membrane does not come in contact with the vial tip. The solution contains no preservatives and must therefore be used within 12 hours of the first opening. For correct storage, accurately close the bottle after use. Do not dispose of the vial in the environment after use. Any severe accident resulting from the use of the medical device must be immediately reported to the manufacturer and the competent authority of the member state in which the event occurred.

PACKAGE: fifteen 0.5 ml preservative-free, single-dose vials. **DOSEAGE:** 1-2 drops per eye every 4-5 hours. We recommend alternating treatment cycles with suspensions of 7 days. Each cycle should not exceed 30 days of treatment.

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