10/15 MONODOSE 0,5 ml - 10/15 SINGLE-DOSE 0,5 ml

Composizione/Composition: Purified Water, *Sea Water (28-30%), Jacobaea Maritima Extract (0,01%), Calendula Officialis Flower Water (0,10%), Centaurea Cyanus Flower Water (0,10%), Helichrysum Italicum Flower Water (0,10%), Euphrasia Officialis Flower/Leaf/Stem Water (0,10%), Hydroxypropylmethylcellulose (0,20%), Sodium Hyaluronate (0,20%). *Isotonic

IT ISOMAR® Occhi Gocce Oculari

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EN ISOMAR® Occhi Eye Drops

INDICATIONS FOR USE: **Isomar Occhi** provides relief from redness, irritation, burning, itching and the sensation of foreign bodies, as well as from tearing and fatigue induced by factors such as: exposure to wind and sunlight, dust, smog, smoke, dry heat, air-conditioned environments, prolonged exposure to computer screens and contact lens wear. **Isomar Occhi** contributes significantly in alleviating the discomfort caused by pollen coming into contact with the conjunctiva by relieving allergic symptoms. **Isomar Occhi** helps with dry eyes and can be used with all types of contact lenses. Application of **Isomar Occhi** prior to fitting contact lenses is recommended for an enhanced lubricating and soothing action.

INSTRUCTIONS FOR USE: to use the single-dose vials correctly, follow the instructions below:

- open the protective packaging and detach a single-dose vial;
- open the single-dose vial by twisting the cap until it is released;
- tilt the head and, holding the lower eyelid gently with a finger, apply 1-2 drops of solution in the conjunctival sac, every 4-5 hours;
- close the eye for a few seconds.

When opened for the first time, the vial can be closed again using the cap and the remaining eye solution can be used over the next 12 hours. If required, the procedure can be repeated for the other eye.

INTENDED PATIENT POPULATION: the medical device is intended for users with no specialist knowledge by observing the manufacturer's instructions for use. The medical device is intended for use in home environments. The user is provided with all the necessary information for the correct and safe use of the device.

CONTRAINDICATIONS: the medical device is not indicated for individuals with known hypersensitivity to the ingredients in the formulation.

WARNINGS: carefully read the information provided by the manufacturer before use. Do not exceed the recommended dose. Discontinue treatment if any undesirable effect occurs. Do not use in conjunction with other topical treatments. Do not swallow. The product is intended for ophthalmic use only. Store inside the box in a dry location, away from sources of light and heat, at a temperature between 5°C and 25°C. Do not pierce or tamper with the packaging. Do not use the product if the package is opened or damaged. Do not use the product after the expiry date. The expiry date refers to the unopened and properly stored product. The product is intended for adult patients. There is no specific data available regarding the use of the product on children and pregnant women. Keep out of sight and reach of children. It is advisable to wash hands thoroughly before applying the solution and to not touch the tip of the dispenser with the fingers. When administering the solution, avoid touching the conjunctival membrane with the tip of the vial. The solution contains no preservatives and must therefore be used within 12 hours of the first opening. For correct storage, accurately close the bottle after use. Do not dispose of the vial in the environment after use. Any severe accident resulting from the use of the medical device must be immediately reported to the manufacturer and the competent authority of the member state in which the event occurred.

PACKAGING: fifteen 0.5 ml preservative-free, single-dose vials or ten 0.5 ml preservative-free, single-dose vials. **DOSEAGE:** 1-2 drops per eye every 4-5 hours. We recommend alternating treatment cycles with suspensions of 7 days. Each cycle should not exceed 30 days of treatment.

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