IT ISOMAR® 20 Flaconcini Decongestionanti

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EN ISOMAR® 20 Decongestant Vials

Hypertonic seawater solution from the area of Cinque Terre.

ISOMAR® Decongestant Vials is a hypertonic solution based on pure seawater with natural decongestant action of nasal tissue. The hypertonic seawater frees nose from excess mucus by osmosis; it is not habit-forming, does not damage the tissue.

ISOMAR® Decongestant Vials is suitable for allergies in general and also caused by pollen. It is recommended for use in children over 6 months, adults and pregnant women.

INDICATIONS: **ISOMAR® Decongestant Vials** is suitable for: fluidizing and improving the drainage of nasal mucus; reducing swelling in the upper air way; eliminating stagnant mucus from nasal passages in colds and allergic rhinitis; relieving symptoms in patients affected by sinusitis and rhino-sinusitis; thoroughly cleansing the nasal cavities removing pollen, dust and smog. **ISOMAR® Decongestant Vials** are also recommended for improving the patency of the nasal passages, restoring ciliary motility and in aerosol therapy for humidifying the upper airways.

FREQUENCY OF USE: use up to a maximum of 3 times a day and for no more than 2 weeks of treatment. Prolonged use could cause dehydration of the nasal mucosa. Do not exceed the recommended dose.

USE: open the sealed up vial turning the cap. Put the mono dose vial in the nostrils of the baby, who is laid down. Press the vial till to have the desired quantity (See picture 1). For the child who use **ISOMAR® Decongestant Vials** by himself and for the adults, the vial must be put in the nostrils, pressing it, head up breathing in and closing the other nostril with the finger (See picture 2).

FOR AEROSOL THERAPY: pour the liquid into the aerosol therapy device, following the instructions for use stated on the appliance.

COMPOSITION: hypertonic sterile seawater.

CONTRAINDICATIONS: do not use **ISOMAR® Decongestant Vials** in infants younger than 6 months old. Do not use in case of epistaxis.

WARNING: sterile solution not suitable for injection. Store in a dry place, away from heat sources at a maximum temperature of 25°C. Do not use concomitantly with other topical treatments. Do not use the same single-dose vial for both nasal hygiene and aerosol therapy. Do not mix with other solutions. Keep the vial out of the reach of children. After opening, the vial must be used immediately and disposed of after use. Failure to check the integrity of the vial may alter the functional characteristics of the product. Do not use beyond the expiration date. Do not leave the empty vials in the environment. During aerosol therapy, we recommend alternating the use of **ISOMAR® Decongestant Vials** with pharmacological treatments. In some cases the product may be slightly irritating.

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