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EN ISOMAR® Spray no gas Baby

ISOMAR® Spray No Gas Baby isotonic solution of sea water (Mediterranean, Cinque Terre, Liguria Italy) for nasal hygiene of children from sixth months of age to adults.

INDICATIONS: the medical device is indicated in case of cold, sinusitis, allergies in general also from pollen and for those who work indoors especially in the presence of dust, useful for those who snore (acute and chronic snoring).

ISOMAR® Spray No Gas Baby with its fluidifying action can be used daily to increase the humidity of the nasal mucous membranes. In the child who has not matured a complete mastery in blowing the nose, it favours a greater patency of the nostrils. In adults, it allows better drainage of nasal mucus. **ISOMAR® Spray No Gas Baby** can be used for correct daily nasal hygiene, in children from the sixth month of life, in pregnant women and during the breastfeeding period. It is not addictive. The mechanical action dispenser, which does not use propellant gases, allows over 400 micronebulisations. **ISOMAR® Spray No Gas Baby** is in a 30ml bottle.

COMPOSITION: demineralised water, sea water (30%).

CONTRAINDICATIONS: Do not use in children under 6 months of age. Do not use in case of hypersensitivity to one or more ingredients.

HOW TO USE: for proper use, remove the cap and, holding both the head and the bottle vertically, insert the opening of the dispensing spout inside the nostril, without sinking excessively into the nasal cavity. Press with two fingers on the base of the dispenser and inhale at the same time as the solution comes out. Repeat the operation with the other nostril. Rinse the dispensing spout with water before replacing the cap. Procedure: 1-2 sprays per nostril up to a maximum of 3 times a day. Allow the solution to act for a few seconds before blowing your nose. Do not exceed the recommended dose.

WARNINGS: does not contain propellant gas. Do not use in conjunction with other topical treatments. Do not exceed the recommended dose. Keep out of the sight and reach of children. Use the product exclusively for its intended use. It can be used on children from 6 months of age and must be applied by an adult. Do not use if package is damaged. Failure to check the integrity of the product could alter its functional characteristics. Do not use the product after the expiry date. Store at a temperature between 5°C and 25°C (41°F - 77°F) away from light and heat sources. Do not dispose of in the environment after use. In the event of a serious accident occurring in relation to the medical device, report it immediately to the Competent Authority of the Member State in which the accident occurred. **ISOMAR®** is also offered in a 100 ml spray version with a nebulising dispenser for nasal hygiene and with the alternative nebuliser (inserted in the case) for ear hygiene. Isomar is also available in sterile, resealable and reusable vials, also suitable for eyewashes and aerosol therapy.

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